

Blood Sample Packaging Instructions For Refrigerated Samples
Prior to packaging gel pack must be refrigerated for at least 24 hours
and ice pack must be in freezer for at least 24 hours

1. Place EDTA blood tube (not supplied) in Styrofoam multi-tube pack with absorbent paper strip and tape closed



2. Wrap Styrofoam multi-tube pack with **REFRIGERATED** Gel Pack



3. Place Styrofoam multi-tube pack/
Gel Pack(item #2) in plastic Ziploc bag



4. Place **FROZEN** ice pack (Koolit) in
bottom of Styrofoam cooler box



5. Place barrier material (ie bubble
wrap (1 inch thick)) on top of **FROZEN**
ice pack



6. Place Ziploc bag (item #3) on top of
barrier material in cooler box (item
#5)



7. Place lid on Styrofoam cooler box
and tape closed, place in shipping box,
tape closed, make sure there is a Live
Cell Test sticker on outside and ship
FedEx **PRIORITY OVERNIGHT** to IBT

